# APFD 201

<b>de</b>	Gebrauchsanweisung Einsatz
<b>cs</b>	Návod k obsluze nástavce
<b>da</b>	Brugsanvisning Indsats
<b>el</b>	Οδηγίες χρήσης Ένθετο
<b>en</b>	Operating instructions Insert
<b>es</b>	Instrucciones de manejo Complemento
<b>et</b>	Kasutusjuhend Element
<b>fi</b>	Käyttöohje Teline
<b>fr</b>	Mode d'emploi Complément
<b>hr</b>	Upute za uporabu Umetak
<b>hu</b>	Használati útmutató Betét
<b>it</b>	Istruzioni d'uso Inserto
<b>lt</b>	Naudojimo instrukcija Naudojimas
<b>lv</b>	Lietošanas instrukcija Ieliktnis
<b>nl</b>	Gebruiksaanwijzing inzet
<b>no</b>	Bruksanvisning Innsats
<b>pl</b>	Instrukcja użytkowania Wkład
<b>pt</b>	Instruções de utilização Complemento
<b>ru</b>	Инструкция по эксплуатации вставки
<b>sk</b>	Návod na použitie Nadstavec
<b>sv</b>	Bruksanvisning Insats

<b>de</b> .....	4
<b>cs</b> .....	12
<b>da</b> .....	20
<b>el</b> .....	28
<b>en</b> .....	36
<b>es</b> .....	44
<b>et</b> .....	52
<b>fi</b> .....	60
<b>fr</b> .....	68
<b>hr</b> .....	76
<b>hu</b> .....	84
<b>it</b> .....	92
<b>lt</b> .....	100
<b>lv</b> .....	108
<b>nl</b> .....	116
<b>no</b> .....	124
<b>pl</b> .....	132
<b>pt</b> .....	140
<b>ru</b> .....	148
<b>sk</b> .....	156
<b>sv</b> .....	164

---

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<b>Notes about these instructions</b> .....	<b>36</b>
Symbols and conventions used in this document.....	36
Definition of terms .....	36
<b>Appropriate use</b> .....	<b>37</b>
Queries and technical problems .....	37
<b>Items supplied</b> .....	<b>38</b>
Load carriers.....	38
Disposal of the packing material .....	38
Optional accessories.....	38
<b>Warnings and safety notes</b> .....	<b>39</b>
<b>Areas of application</b> .....	<b>40</b>
Colour coding.....	40
Preparing the load .....	40
Preparing the insert.....	40
Pre-cleaning breathing masks.....	40
Dismantling the breathing mask.....	41
Arranging breathing masks.....	41
Completely loading the insert.....	42
Placing the insert in the load carrier .....	42
After reprocessing .....	42

## en - Notes about these instructions

---

### Symbols and conventions used in this document

#### Warnings

 Warnings contain information which is important for safety. This alerts you to the potential danger of injury to people or damage to property.  
Read these warning notes carefully and observe the procedural instructions and codes of practice they describe.

#### Notes

Notes provide information of particular importance that must be observed.

#### Additional information and comments

Additional information and comments are contained in a simple frame.

#### Operating steps

Operating steps are indicated by a black square bullet point.

##### Example:

■ Select an option using the arrow buttons and save your choice with *OK*.

##### Example:

■ Select an option.

#### Display

Display text can be identified from the special font.

##### Example:

Save.

### Definition of terms

#### Cleaning machine

In these operating instructions, the dishwasher or the washer-dishwasher is referred to as “the machine”.

#### Load items

The term “load items” is used wherever the items to be processed are not defined in any further detail.

#### Wash water

The term “wash water” is used for the mixture of water and process chemicals.

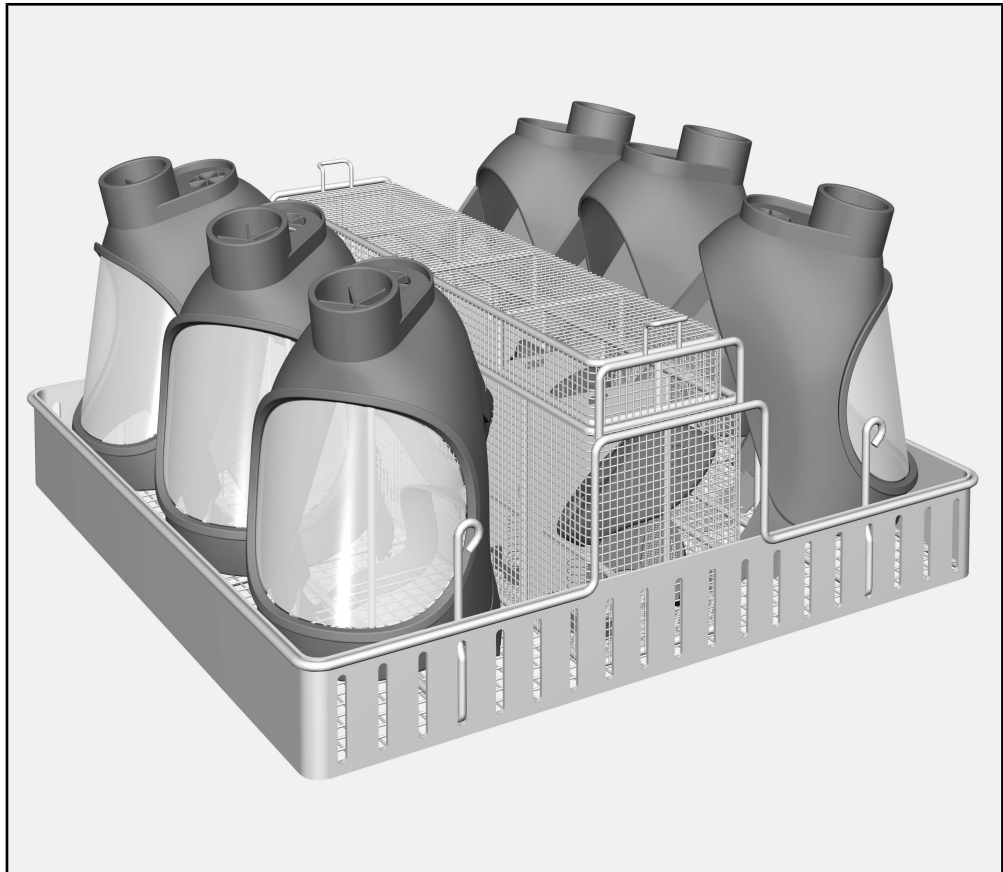
The APFD 201 insert can be used to reprocess machine-reprocessable breathing masks in a Miele dishwasher or a machine. Follow the operating instructions for your dishwasher or the machine as well as the instructions issued by the manufacturer of the breathing masks.

The APFD 201 insert can be used together with the A 151 lower basket.

### **Queries and technical problems**

If you have any queries or technical problems that you would like to discuss please contact Miele. Contact details can be found at the back of the Operating Instructions for your washer-disinfector or go to [www.miele-professional.com](http://www.miele-professional.com).

### Load carriers

- APFD 201 insert, height 219 mm, width 470 mm, depth 480 mm

### Items supplied

- 2 x compartment for small parts, two-piece, divided into three parts

### Disposal of the packing material

The packaging is designed to protect against transportation damage. The packaging materials used are selected from materials which are environmentally friendly for disposal and should be recycled.

Recycling the packaging reduces the use of raw materials in the manufacturing process and also reduces the amount of waste in land-fill sites.

### Optional accessories

The following and other accessories are available to order from Miele:

- APFD 202, 6 adapters for loading specially shaped air regulators, e.g. from Interspiro.

Read the operating instructions carefully before using this insert.  
Keep these operating instructions in a safe place.

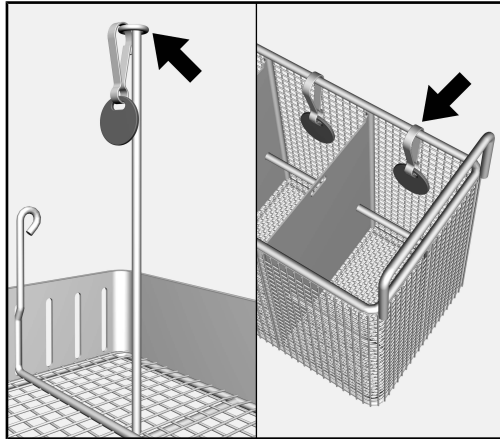
 Please also read the operating instructions for your washer-disinfector and pay particular attention to the Warning and Safety instructions.

- ▶ The inserts are only approved for use in the applications stated in the operating instructions. Applications that go beyond those listed and use of the inserts with additional components from the Miele range of components are described in the operating instructions of the components or are to be agreed with Miele. Conversions, modifications and any other use are not permitted.
- ▶ Prior to being used for the first time, new inserts must undergo thorough cleaning in the washer-disinfector without load items.
- ▶ Inspect all mobile units, baskets, modules and inserts as described in the “Maintenance” section of the operating instructions for your machine.
- ▶ Only items which have been declared by their manufacturer as suitable for machine reprocessing may be processed. The manufacturer's specific reprocessing instructions must be observed.
- ▶ Follow the warnings and instructions provided by the breathing apparatus manufacturer.
- ▶ Any seals on the load items should be protected against prolonged contact with wash water. Therefore, always close unused connections with protective caps or blind covers.
- ▶ The last rinse, as a minimum, must be carried out with DI water.
- ▶ Disposable items must not be reprocessed.

### Colour coding

The small parts of a dismantled breathing mask are reprocessed in a compartment of the small-part basket. Colour-code the compartments and the load item carriers to ensure that the small parts can be assigned to the right breathing mask after reprocessing.

Coloured tags (2 in each colour) are provided with the inserts for this purpose.

For each colour, attach one tag to the load item carriers and one tag to the small-part basket compartments.

When using other systems for identification, ensure that these are suitably chemical-resistant and temperature-resistant.

### Preparing the load

#### Preparing the insert

- Place the insert on the worktop and ensure that it is securely positioned.
- Lift the top insert out of the small-part basket.
- Place the top insert next to the insert and open the lid.

The insert holders and the adjacent compartments of the small-part basket must have the same colour.

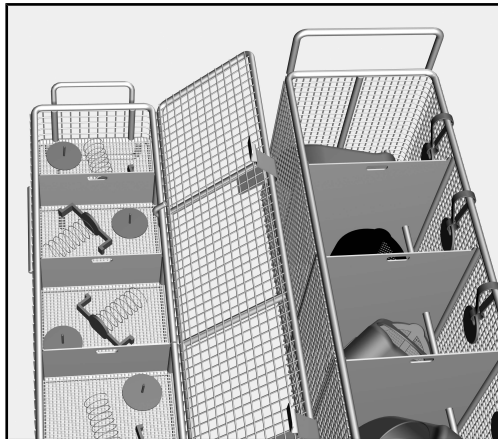
#### Pre-cleaning breathing masks

- Check whether the breathing mask requires manual pre-cleaning, e.g. to remove soot.
- If required, carry this pre-cleaning out in line with the manufacturer's instructions.

### Dismantling the breathing mask


- Dismantle the breathing mask in line with the manufacturer's instructions.

- Place the small parts in a compartment of the top insert of the small-part basket.
- Place the inner mask in the compartment in the small-part basket with the curved hollow side facing down.

Note the colour-coding for the small-part basket compartment to place the breathing masks in the same colour in the insert.

### Arranging breathing masks

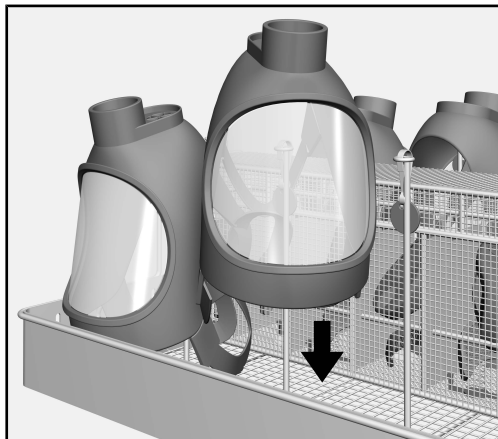- Place the breathing masks on the holders on the APFD 201 insert. Ensure that the colour coding for the holder corresponds to that for the compartment in the small-part basket.
- Arrange the masks in such a way that as little wash water as possible can collect in curved hollow areas.

Risk of damage to visors.

Touching visor panels can become scratched during the loading and unloading process.

Arrange the masks in such a way that the visor panels do not touch.

## en - Areas of application

---

### **Completely loading the insert**

- Equip the remaining positions of the insert in the same way.
- Close the lid on the top insert of the small-part basket.
- Place the top insert on the small-part basket.

### **Placing the insert in the load carrier**

- Lift the insert at the side handles.
- Place the insert in the load carrier of the machine.

### **After reprocessing**

It is essential that you pay attention to the “Application technology” chapter in the operating instructions for the dishwasher or the machine and the instructions specified by the manufacturer of the breathing masks.

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